

inora

Nasogastric tube fixation



INORA AS
P.box 92
2302 HAMAR
NORWAY

post@inora.no
www.inora.no

Org. nr. 992 804 696

inora
converting ideas into solutions

eario

Nasogastric tube fixation

EARIO safely fixates the nasogastric tube to the patient ear, prevents skin irritation from tape and stops the tube from kinking.

- No adhesive and skin irritation
- No kinking
- Easy to use
- Individual adjustment
- Two sizes: 8 and 10 Fr.



INSTRUCTIONS ON USE:

Preparations:

- Open the package and read the instructions before use.
- Product is clean but not sterile.

Use:

- Place the EARIO on the patient's ear, and fixate the tube under the two fixation clips.
- Tighten EARIO around the ear by pressing the two fixation clips towards each other.
- Make sure to use the right sized EARIO (8 or 10 fr.) for the desired tube to avoid kink.

After use:

- EARIO is to be used for one patient only and must be disposed after use.

Place EARIO gently around the patient's ear from above. Pull the tube carefully into the two fixation clips and press firmly into place.

Make sure to use the correct size EARIO for the intended tube. The number shown on top of the clips indicates the size in Fr.

Press the two clips together to tighten the grip around the ear for an individual fit.

If the tube is hanging too much down from the patient's nose, just pull carefully on the tube from the other side of EARIO

